MEDI PLAST [®]

BamBa™ PREMATUR

🇸🇪 Mediplast AB Kantyxegatan 29, SE-213 76 Malmö, Sweden Tel.: +46 40 671 23 00 / www.mediplast.com
☒ 📖 CE

EN

BamBa™ PREMATUR

1-piece transparent and drainable ostomy pouch with filter for premature infants (≤ 900 g)

Product description
BamBa™ Prematur is a drainable ostomy pouch for premature infants up to about 900 g (1 lb, 15.75 oz). It has a built-in filter and a skin barrier comprising a soft, skin-friendly hydrocolloid that can be cut for optimal fit. The front is transparent and the back is covered by a soft, non-woven layer for a well-adapted feel on the infant’s skin. One box contains 20 BamBa™ Prematur pouches with attachable, soft clamps for closing the pouch and protective covers for the filter.

Instructions
Adapting the hole in the skinwafer
To avoid leakage problems and irritated skin it is important that the skin wafer fit snugly around the stoma. If you are unsure of the stoma size, use a measuring guide to control the size. Use the cutting guide on the skin wafer and draw the stoma shape on the protective paper. Before cutting, blow some air into the pouch to separate the plastic layers, which decreases the risk of cutting into the pouch. Adjust the hole in the wafer using small curved scissors.
Preparing the skin
Wash the skin around the stoma with lukewarm water. Avoid oil-based soaps or creams since they may decrease the skin wafers adhesive properties. If soap is used it should be unscented.
Allow the skin to air-dry. The skin should be completely dry before attaching the skin wafer around the stoma.
Attaching the pouch
A body-temperature hydrocolloid adhesive always fastens best. Therefore, warm the skin barrier between your hands for a while before attaching it around the stoma. The pouch is transparent to make it easier to apply the skin wafer correctly around the stoma. Remove the protective paper from the skin wafer and attach to the skin around the stoma. Gently rub the skin wafer to make it attach it even better.
Removing the pouch
Start by dabbing with plenty of lukewarm water around skin wafers edges; this will saturate the hydrocolloid and make it easier to remove. Gently roll off the wafer downwards. Avoid using oil, alcohol or solvents as it can affect how well the next skin barrier attaches to the skin
Opening or closing the outlet
Cut open the pouch’s outlet. Remove the protective paper and place it at the outlet on the front of the pouch. Fold the clamp two turns up against the front of the pouch. Fold in both ends of the clamp over so they meet. Reverse the order to open.

Waste handling
Used pouches are drained, packaged and dispose with household waste.
Warnings and precautions
Store in a cool and dry place, but never in the refrigerator.
Do not use if the packaging is damaged or after the expiry date.

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