




Medibender

A bending instrument for Medioplast Metal Suction Tips.
Stainless Steel, non sterile, can be autoclaved.

Art.No.	Description	Ward/Trsp.box
6066501	4.0, 3.0, 2.0, 1.4 and 1.0 mm	1/1

How to use it:

1. Grip the instrument with the thumb on the Medioplast engraving and with the holes visible
2. Take the wanted size of metal suction tip
3. Put the metal suction tip through the appropriate sized hole
4. Bend the metal suction tip gently in the length direction of the instrument to the angle wanted

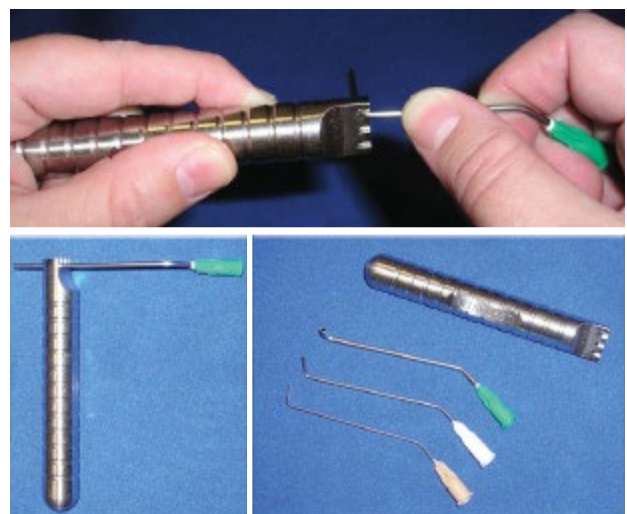

Manufacturer: Medioplast AB / www.medioplast.com