 MEDI PLAST[®]	
	
	
SnapOn™	
UroBa®	
🏠 Mediplast AB Kantyxegatan 29, SE-213 76 Malmö, Sweden Tel.: +46 40 671 23 00 / www.mediplast.com ⓧ ⓘ ⒸⒺ	

EN

SnapOn™ UroBa®
2-piece ostomy pouch with a non-return valve and drain

Product description
SnapOn™ UroBa® is a 2-piece, drainable ostomy pouch for urostomy with a built-in non-return valve, so that the content does not flow back onto the stoma, and a soft drain fixture for easy emptying. SnapOn™ pouches have an attachment ring with a special finger grip at the top of the ring. It allows you to easily detach the pouch from the attachment ring on the barrier. SnapOn™ pouches fit all SnapOn™ Skin Barriers, even convex. The pouch is covered by a soft, non-woven layer and has an openable front with an inspection window and comes in sizes Midi (300 ml) and Maxi (480 ml) with two different ring diameters: 45 mm and 57 mm. One box contains 30 SnapOn™ UroBa® pouches and includes night connectors. NOTE! SnapOn™ UroBa® Ostomy Pouches must be used with SnapOn™ Skin Barriers.

Instructions
Applying the Skin Barrier: Please see the instructions for SnapOn™ Skin Barrier
Fastening the pouch to the skin-barrier attachment ring
Place both hands around the skin protection ring’s retractable attachment ring, starting at the lower side of the ring. Then press the rings together making sure they connect to each other correctly, continue upwards until the rings are fully in position. Pull the pouch’s lower section down and check that it sits firmly on the ring.
Changing bandages
Always have a compress, or similar, on hand to capture any leakage/residues. Keep one hand on the skin barrier to hold it firmly in place. Take hold of the attachment ring’s upper loop and pull the pouch forward and outward until it loosens.
Emptying the pouch
Hold the pouch outlet away from the body and pull the plug that is attached to the outlet upwards. Then, lower the drain/outlet over the toilet and empty. Wipe off the outlet and reset the stopper. Make sure to press it all the way down to the bottom of the outlet.
Night connector
Each package contains connectors. These are used when connecting the pouch outlet to a night pouch. Before connecting the ostomy pouch, allow some liquid to remain in it in order to prevent negative pressure. Connect the night pouch to the narrower part of the connection piece and open the pouch outlet by pushing all the way down to the bottom of the wider part of the connector.

Waste handling
Drain the used dressing, place in the wrapper and dispose with household waste.
Warnings and precautions
Store in a cool and dry place, but never in the refrigerator.
Do not use if the packaging is damaged or after the expiry date.

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