



Indication: As protection against wound damages caused by friction and body fluids (incontinence). Waterproof fixation of primary dressings e.g. swabs, alginate dressings or polyurethane foam dressings.

## Film Dressing on roll

MEDIPLAST® offers non-sterile transparent film dressings on roll. Cut to size with maximum flexibility. Smooth and formable after body contours. The elasticity enables movement without hurting the skin. The product is non-sterile and should not be used on damaged skin, open wounds or IV-entering. The roll is single-packed in a box.

Length: 10 meter


Article No	Description	Pcs/ ward box	Pcs/ trsp.box
60701005	5 cm	1	48
60701010	10 cm	1	24
60701015	15 cm	1	24

Do like this:

Cut to desired length.

Remove the white protective paper and apply the transparent film to the skin.

Peel off the upper transparent protective film by pulling it in the opposite direction of the indicator arrows.

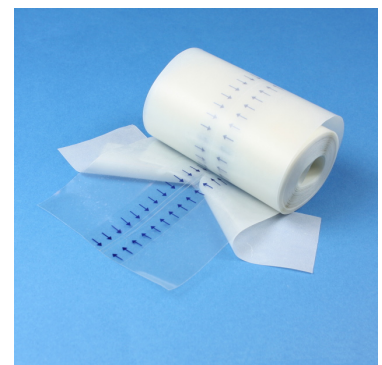

Ward box = min order qty