

**ME
COVERS®**
ADDS INTEGRITY



MECOVERS INTEGRITY PROTECTION

Provides a better experience

The MeCovers skirt is an awaited solution that increases comfort and dignity for the patient during medical examinations of the lower body.

By being offered the possibility to cover up and restrict exposure, every patient can enter her or his examination with a sense of dignity.

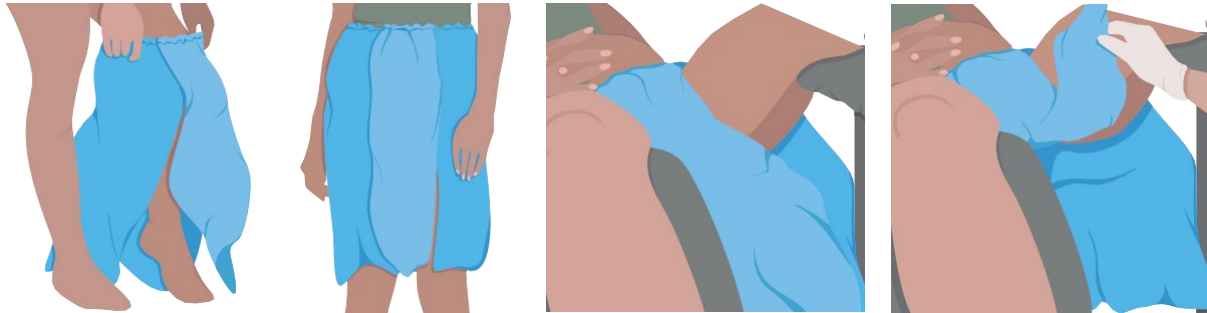
MeCovers is a unisex product developed to change and improve the health care experience for both women and men by adding a layer of integrity and control.

MeCovers is intended to be used primarily for gynaecological, urological and uro-gynaecological examinations. Overall it can be used in any kind of medical or health care environment where only a specific part of the body needs to be fully exposed.

MeCovers is a disposable product held together with an elastic band. Put the skirt on by pulling it over your head, or step into it feet first. The fly's position is adjusted to suit the needs of the specific examination.

Art.No.	Description	Ward/Trsp.box
K6600013	MeCovers skirt	10/100
K6600014	MeCovers skirt	150/150
D6600010	Dispenser for MeCovers skirt	1/4

Ward box = min order qty

**Note: Important information**

- MeCovers is not sterile and can therefore not be used on damaged skin or in open surgery. Damaged skin on patient should be bandaged before the skirt is put on.
- Minor static charge may occur. Do not use ESD-sensitive electronic instruments at the same time as the skirt.
- The skirt is made in one size. Health care staff must provide instructions to the patient: If the skirt is too long, the patient should hold it up when he or she moves.

Medical device 100% polypropylene

NOTE! Risk of melting when close to high heat or flame

