 MEDI PLAST[®]	
	
	
SnapOn™	
KoloBa™	
🏠 Mediplast AB Kantyxegatan 29, SE-213 76 Malmö, Sweden Tel.: +46 40 671 23 00 / www.mediplast.com	

EN

SnapOn™ KoloBa™
2-piece, closed, colostomy pouch with extra protected filter

Product description
SnapOn™ KoloBa™ is a closed, 2-piece, ostomy pouch for colostomy, with a carbon filter with a unique extra partition wall with air channels. This substantially reduces the risk of the filter becoming covered or clogged. SnapOn™ pouches have an attachment ring with a special finger grip at the top of the ring. It allows you to easily detach the pouch from the attachment ring on the barrier. SnapOn™ pouches fit all SnapOn™ Skin Barriers, even convex.
The pouch is covered by a soft, non-woven layer and has an openable front with an inspec- tion window and comes in sizes Mini (130 ml), Midi (230 ml) and maxi (310 ml) with two different ring diameters: 45 mm and 57 mm.
One box contains 30 KoloBa™ pouches and filter cover that protects in the shower and bath. NOTE! SnapOn™ KoloBa™ Ostomy Pouches must be used with SnapOn™ Skin Barriers.

Instructions
Applying the Skin Barrier: Please see the instructions for SnapOn™ Skin Barrier
Fastening the pouch to the skin-barrier attachment ring
Place both hands around the skin protection ring’s retractable attachment ring, starting at the lower side of the ring. Then press the rings together making sure they connect to each other correctly, continue upwards until the rings are fully in position. Pull the pouch’s lower section down and check that it sits firmly on the ring.
Changing bandages
Always have a compress, or similar, on hand to capture any leakage/residues. Keep one hand on the skin barrier to hold it firmly in place. Take hold of the attachment ring upper loop and pull the pouch forward and outward until it loosens.

Waste handling Used pouches are drained, packaged and dispose with household waste. Warnings and precautions Store in a cool and dry place, but never in the refrigerator. Do not use if the packaging is damaged or after the expiry date.
--

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