

Traditional G-Tubes

Adjustable Length Gastrostomy Tubes



Balloon



Capsule Dome



Capsule Monarch®

Reduce Leaks

Improve Comfort + Safety

Ease of Placement


Exclusive Capsule Technology




Innovating. Educating. Changing Lives.™

AMT Traditional Length G-Tubes


An Advantage in Enteral Nutrition

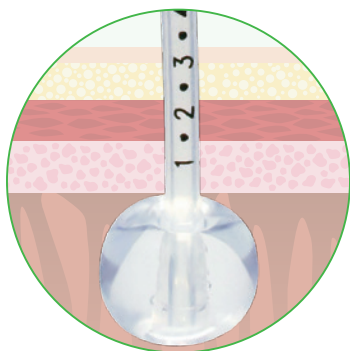

Balloon

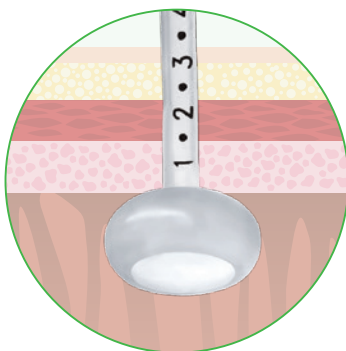

Capsule Dome

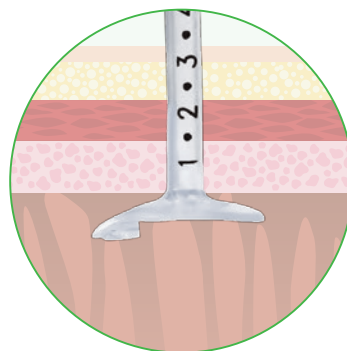

Capsule Monarch®

AMT Balloon G-Tube

Reduce Leaks

- Exclusive AMT Balloon expands horizontally to provide superior stoma coverage¹

Improve Comfort + Safety

- Balloon protects the distal tip of the tube, lowers the risk for gastric ulceration
- Enhanced External Retention Ring, minimizes contact and allows skin to breathe

Ease of Placement

- Often placed in the office or on-site
- Tapered tip design for easier placement
- Guidewire compatible
- Radiopaque for x-ray confirmation

Balloon G-Tube

Legacy and ENFit®

12F – 24F



ENFit®
Ready

Exclusive AMT Capsule Technology

Capsule Dome. Capsule Monarch®.


Ease of Placement

- Encapsulated internal bolster, self-lubricating to minimize discomfort during placement
- Often placed in the office or on-site
- Radiopaque for x-ray confirmation
- Guidewire compatible


Non-Balloon Internal Bolster

- Great alternative for patients who cannot sustain a balloon device
- Capsule Dome is durable + secure
- Capsule Monarch® internal bolster sits flush against the gastric wall

Stages of Capsule Bolster Deployment

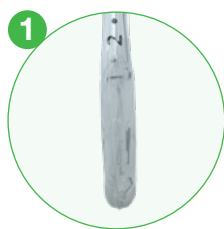

Encapsulated Bolster Inserted into Established Stoma

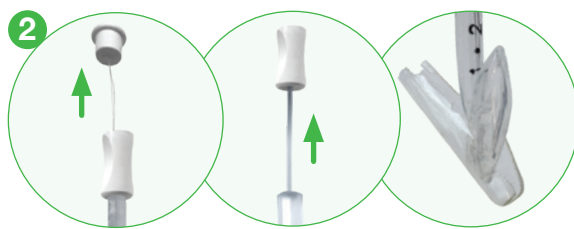


Tether Pulled to Split Capsule Material, Stiffener Removed



Capsule Material Falls Away and Dissolves Rapidly


Capsule Dome G-Tube

Legacy and ENFit®
20F, 24F

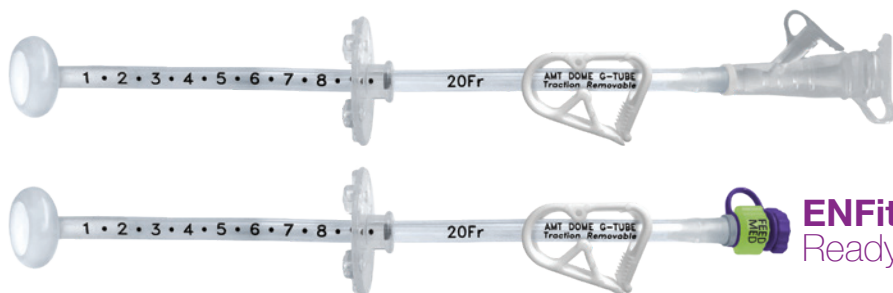

Capsule Monarch® G-Tube

Legacy and ENFit®
12F, 14F, 18F, 20F

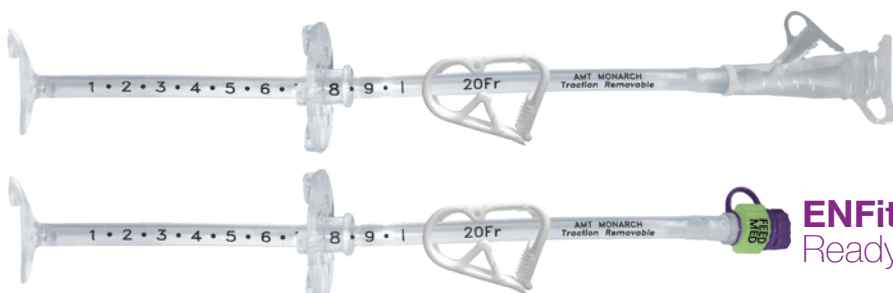

AMT Traditional Length G-Tubes

Order Numbers + Accessories

Balloon G-Tube

AMT	ORDER NUMBERS	ENFit® ORDER NUMBERS
12F	7-1210	7-1210-I
14F	7-1410	7-1410-I
16F	7-1610	7-1610-I
18F	7-1810	7-1810-I
20F	7-2010	7-2010-I
24F	7-2410	7-2410-I

AMT Balloon G-Tube kits are pouched (sterile) and sold 1/box.

Legacy (non-ENFit®) Balloon G-Tube kits are pouched with a pinch clamp.

ENFit® Balloon G-Tube kits are pouched with an ENFit® Single-Port to Y-Port Adapter and a pinch clamp.

Balloon Fill Volumes

The recommended water fill volume is clearly printed on each Balloon G-Tube. Consult your healthcare provider before adjusting the fill volume.

Size	Min	Rec.	Max
12F	2 ml	2.5 ml	3 ml
14F	3 ml	4 ml	5 ml
16F	4 ml	6 ml	8 ml
18F	6 ml	8 ml	10 ml
20F	7 ml	10 ml	15 ml
24F	7 ml	10 ml	15 ml



Capsule Dome G-Tube

AMT	ORDER NUMBERS	ENFit® ORDER NUMBERS	GUIDEWIRE†
20F	9-C2020	9-C2020-I	--
	9-CG2020	9-CG2020-I	0.038" x 90 cm
24F	9-C2420	9-C2420-I	--
	9-CG2420	9-CG2420-I	0.038" x 90 cm

AMT Capsule Dome G-Tube kits are pouched (sterile) and sold 1/box.

Legacy (non-ENFit®) Capsule Dome kits are pouched with a pinch clamp.

ENFit® Capsule Dome kits are pouched with an ENFit® Single-Port to Y-Port Adapter and a pinch clamp.

† Guidewire is included in a separate pouch (sterile).

Replacement Adapters + Accessories

Device	Size	Description	Order #
Capsule Dome, Capsule Monarch®			
	12F	Purple Y-Port	4-2012
	14F	Purple Y-Port	4-2014
	15F	Clear Y-Port XL	4-2015
	20F	Clear Y-Port XL	4-2020
	24F	Clear Y-Port XL	4-2024
	12F	ENFit® Single-Port	4-2012-ISOSAF
	14F	ENFit® Single-Port	4-2014-ISOSAF
	20F	ENFit® Single-Port	4-2020-ISOSAF
	24F	ENFit® Single-Port	4-2024-ISOSAF
Balloon, Capsule Dome, Capsule Monarch®			
	Universal ENFit®	ENFit® Single-Port to Y-Port	4-0005-ISOSAF
Avoid Disconnections with Tube Securement			
	Medium CINCH® 4F - 17F	Box of 10	CINCH417M-10
		Box of 50	CINCH417M-50
	Large CINCH® 9F - 30F	Box of 10	CINCH930L-10
		Box of 50	CINCH930L-50
	Legacy Connections	AMT CLAMP™ Box of 10	4-3000

Replacement Adapters are individually pouched and sold 10/box. A pinch clamp is included in each pouch.

Capsule Monarch® G-Tube

AMT	ORDER NUMBERS	ENFit® ORDER NUMBERS	GUIDEWIRE†
12F	9-C1210	9-C1210-I	--
	9-CG1210	9-CG1210-I	0.018" x 70 cm
14F	9-C1410	9-C1410-I	--
	9-CG1410	9-CG1410-I	0.018" x 70 cm
18F	9-C1810	9-C1810-I	--
	9-CG1810	9-CG1810-I	0.038" x 90 cm
20F	9-C2010	9-C2010-I	--
	9-CG2010	9-CG2010-I	0.038" x 90 cm

AMT Capsule Monarch® G-Tube kits are pouched (sterile) and sold 1/box.

Legacy (non-ENFit®) Capsule Monarch® kits are pouched with a pinch clamp.

ENFit® Capsule Monarch® kits are pouched with an ENFit® Single-Port to Y-Port Adapter and a pinch clamp.

† Guidewire is included in a separate pouch (sterile).

Jälleenmyyjä:

MEDI PLAST

Medioplast Fenno Oy

PL 153, 01531 Vantaa

Puh: 09 276 360

asiakaspalvelu@mediplast.com

www.mediplast.com



Traditional G-Tubes

For a complete list of products visit us at www.AppliedMedical.net
800 869 7382 | CS@AppliedMedical.net | ICS@AppliedMedical.net



Applied Medical Technology, Inc.
 8006 Katherine Boulevard
 Brecksville, OH 44141
www.AppliedMedical.net
 An ISO 13485 Registered Company
 Made In USA



© 2021 Applied Medical Technology, Inc. C-4496-F

Patents: <http://www.AppliedMedical.net/Patents/> | Trademarks: <http://www.AppliedMedical.net/Trademarks/>

ENFit® is a registered trademark of GEDSA. | 1. Based on internal testing of 12F devices