

G-JET[®] Button

Low Profile Gastric-Jejunal Enteral Feeding Tube



Reduce Clogging

Anti-Kink Technology

AMT Exclusive Balloon

Improve Comfort




Innovating. Educating. Changing Lives.™

What Makes A Better G-J Button:

Same French Size, Larger Lumens DESIGNED TO REDUCE CLOGGING


Gastric Profile

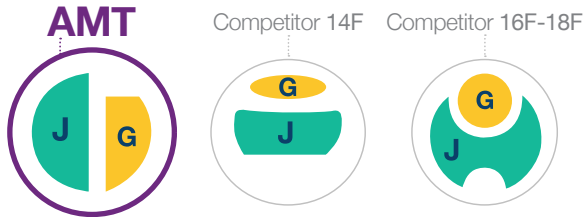

Gastric Lumen Comparison¹

AMT 14F is **58% Larger** than Competitor 14F
AMT 16F is **41% Larger** than Competitor 16F
AMT 18F is **34% Larger** than Competitor 18F

Jejunal Profile

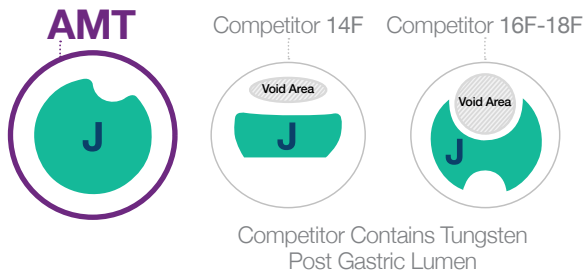

Jejunal Lumen Comparison¹

AMT 14F is **56% Larger** than Competitor 14F
AMT 14F is **56% Larger** than Competitor 16F
AMT 16F is **131% Larger** than Competitor 16F
AMT 16F is **81% Larger** than Competitor 18F
AMT 18F is **172% Larger** than Competitor 18F
AMT 18F is **57% Larger** than Competitor 22F

SUCCESS STORIES

My G-JET[®] Lasts Longer

The length of time between routine exchanges has significantly increased since I switched to the G-JET[®]. Interventional radiology is now replacing my G-JET[®] every 4-5 months... because it can last!



Amanda's G-JET[®]

AppliedMedical.net/Amandas-G-JET

An Advantage in Enteral Nutrition

Anti-Kink. Exclusive AMT Balloon. Glow Green™ Technology.

Reduce Leaks, Superior Comfort

1 Anti-Reflux Valve

Prevents backflow from the G + J ports†

2 Tapered Transition

For a more secure fit

3 Exclusive AMT Balloon



Apple-shaped, expands horizontally
Creates a superior gastric seal²

Soft + Flexible


Medical grade silicone is gentle to the skin and easy to clean around

Four Point Stabilization

Reduces motion that can cause granulation tissue

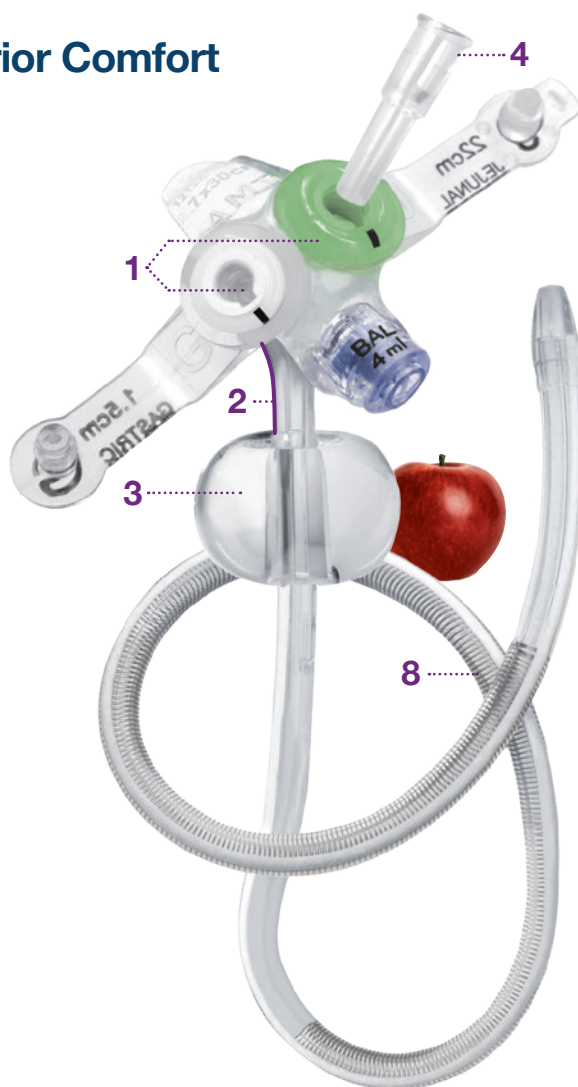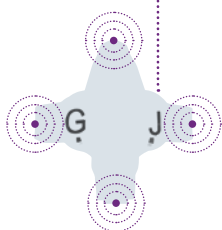

Safety + Innovation

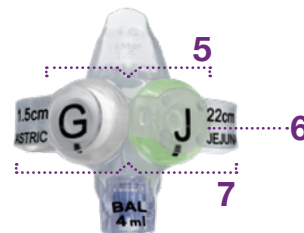

4 Guidewire Adapter

For ease of placement

5 Mutually Exclusive G+J

Lowers risk of misconnecting feed and extension sets

6 Glow Green™ J-Port

Easier nighttime feeds
Feed the Green!

7 Dual Safety Straps

Allow independent access to the G and/or J port

8 Anti-Kink Technology

16F and 18F sizes offer spring reinforcement to maintain flow

Mutually Exclusive Ports Leading the Way in Patient Safety

Accidentally feeding the stomach instead of the jejunum can be dangerous! Mutually exclusive **GASTRIC** and **JEJUNAL** ports are designed to reduce the risk of misconnections. Different feed/extension sets are required for each port, so it is imperative to maintain adequate stock of both sets. Always remember to **Feed the Green!**



Gastric G-Port

Jejunal J-Port



3/4 Turn Clockwise

1/4 Turn Clockwise

6-1222 Extension Set

8-1257G Feed Set

Proper Fit Every Time

To ensure a proper fit, it is crucial to measure the stoma length using the AMT Balloon Stoma Measuring Device

Balloon Stoma Measuring Device	Order #	Packaged
Guidewire Compatible	1804	10/Box
Non-Guidewire Compatible	1704	10/Box

G-JET® Button

G	J	14 F	16 F	18 F
LENGTH		LEGACY ORDER NUMBERS*		
1.0 cm	15 cm	GJ-1410-15	GJ-1610-15	
	22 cm	GJ-1410-22	GJ-1610-22	
1.2 cm	15 cm	GJ-1412-15	GJ-1612-15	
	22 cm	GJ-1412-22	GJ-1612-22	GJ-1812-22
	30 cm	GJ-1412-30	GJ-1612-30	GJ-1812-30
1.5 cm	15 cm	GJ-1415-15	GJ-1615-15	
	22 cm	GJ-1415-22	GJ-1615-22	GJ-1815-22
	30 cm	GJ-1415-30	GJ-1615-30	GJ-1815-30
	45 cm	GJ-1415-45	GJ-1615-45	GJ-1815-45
1.7 cm	15 cm	GJ-1417-15	GJ-1617-15	
	22 cm	GJ-1417-22	GJ-1617-22	GJ-1817-22
	30 cm	GJ-1417-30	GJ-1617-30	GJ-1817-30
	45 cm	GJ-1417-45	GJ-1617-45	GJ-1817-45
2.0 cm	22 cm	GJ-1420-22	GJ-1620-22	GJ-1820-22
	30 cm	GJ-1420-30	GJ-1620-30	GJ-1820-30
	45 cm	GJ-1420-45	GJ-1620-45	GJ-1820-45
2.3 cm	30 cm	GJ-1423-30	GJ-1623-30	GJ-1823-30
	45 cm	GJ-1423-45	GJ-1623-45	GJ-1823-45
2.5 cm	30 cm	GJ-1425-30	GJ-1625-30	GJ-1825-30
	45 cm	GJ-1425-45	GJ-1625-45	GJ-1825-45
2.7 cm	30 cm	GJ-1427-30	GJ-1627-30	GJ-1827-30
	45 cm	GJ-1427-45	GJ-1627-45	GJ-1827-45
3.0 cm	45 cm	GJ-1430-45	GJ-1630-45	GJ-1830-45
3.5 cm	45 cm	GJ-1435-45	GJ-1635-45	GJ-1835-45
4.0 cm	45 cm	GJ-1440-45	GJ-1640-45	GJ-1840-45
4.4 cm	45 cm	GJ-1444-45	GJ-1644-45	GJ-1844-45
5.0 cm	45 cm	GJ-1450-45	GJ-1650-45	GJ-1850-45
5.5 cm	45 cm	GJ-1455-45	GJ-1655-45	GJ-1855-45
6.0 cm	45 cm	GJ-1460-45	GJ-1660-45	GJ-1860-45

The G-JET® Button is sold 1/box (sterile).

NOTE: G-JET® devices should never be cut to custom lengths.

Guidewire available (180 cm): Order Number 4-GW-5

*Add "-I" for ENFit® G-JET® Button kits.

Balloon Fill Volumes

The recommended water volume is clearly printed on the fill valve of each G-JET® Button. Consult your healthcare provider before adjusting the fill volume.

FRENCH SIZE	MIN	REC.	MAX
14F	3 ml	4 ml	5 ml
16F	3 ml	4 ml	5 ml
18F	7 ml	8 ml	9 ml



Äterförsäljare:

MEDI PLAST

Mediplast AB

Box 1004, 212 10 Malmö

T 020-78 80 35

info.se@mediplast.com

www.mediplast.com



G-JET® Button

Extension + Feed Sets Included in Kit




ADAPTERS	RIGHT ANGLE CONNECTORS			STRAIGHT CONNECTORS		
	2"	12"	24"	2"	12"	24"
COMPATIBLE GASTRIC EXTENSION SETS						
Bolus		6-1221	6-2421	6-0211	6-1211	6-2411
Clear Y-Port		6-1222	6-2422	6-0212	6-1212	
ENFit® Y-Port		6-1222-ISOSAF				
COMPATIBLE JEJUNAL FEED SETS†						
Purple Y-Port		8-1257G	8-2455			
ENFit® Y-Port	8-0222-ISOSAF	8-1255-ISOSAF	8-2455-ISOSAF	8-0212-ISOSAF		
Bolus		8-1221	8-2421	8-0211	8-1211	8-2411
ENFit® Single Port		8-1221-ISOSAF	8-2421-ISOSAF	8-0211-ISOSAF	8-1211-ISOSAF	8-2411-ISOSAF
Clear Y-Port XL	8-0222	8-1222	8-2422	8-0212	8-1212	

Legacy Included

ENFit® Included

†G-JET® Jejunol Feed Sets are compatible with Mic-Key® devices. Extension + Feed sets come packaged as 10/box.

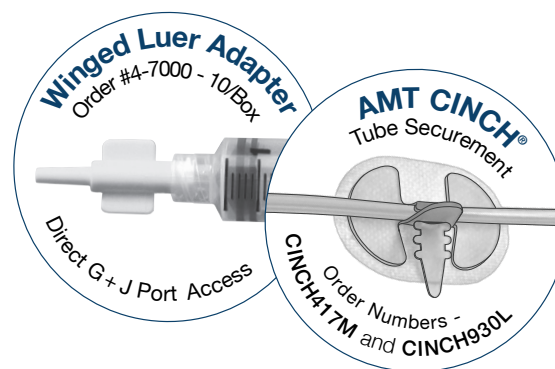

For a complete list of products visit us at www.AppliedMedical.net
800 869 7382 | CS@AppliedMedical.net | ICS@AppliedMedical.net



Applied Medical Technology, Inc.
8006 Katherine Boulevard
Brecksville, OH 44141
www.AppliedMedical.net
An ISO 13485 Registered Company
Cleared For Initial Placement
MR Conditional
Made In USA



© 2021 Applied Medical Technology, Inc. C-4497-J

Patents: <http://www.AppliedMedical.net/Patents/> | Trademarks: <http://www.AppliedMedical.net/Trademarks/>

*The Anti-Reflux Valve is opened when the feed/extension set is locked into place. | 1.Claim based on internal testing of 14F-22F devices. | 2.Claim based on internal testing of 16F devices. Mic-key® is a registered trademark of Avent, Inc. | ENFit® is a registered trademark of GEDSA.