


MiniONE® Button

Low Profile Balloon Gastrostomy Tube

Replacing AMT MiniONE® Balloon Button*

To ensure a proper fit, it is crucial to measure the stoma length using the AMT Balloon Stoma Measuring Device (BSMD). This is not a substitute for the DFU/IFU (Directions for Use/Instructions for Use). 

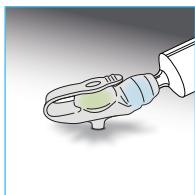


DEFLATE BALLOON

Deflate the existing device using a Luer syringe or Luer Lock syringe.




REMOVE DEVICE

Remove the device, gently pull up on the device with slow and steady pressure.

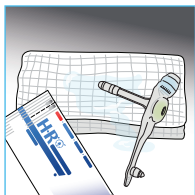


PREPARE KIT

Prepare the new device, apply a water soluble lubricant to the device.

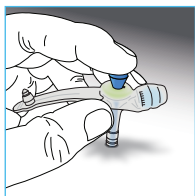

INSERT DEVICE

Insert the device into the stoma site, until it is flush against the skin. Remove the blue stiffener.

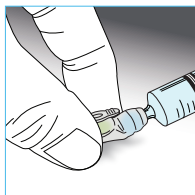

FILL BALLOON

Fill the balloon to the recommended fill volume, which is printed on the balloon fill valve.




Your ONE Source for Enteral Nutrition
Devices. Accessories. Resources. Support.

MiniONE® Button

Low Profile Balloon Gastrostomy Tube

Legacy + ENFit®

Available MiniONE® Family Sizes: Balloon: 12F – 24F

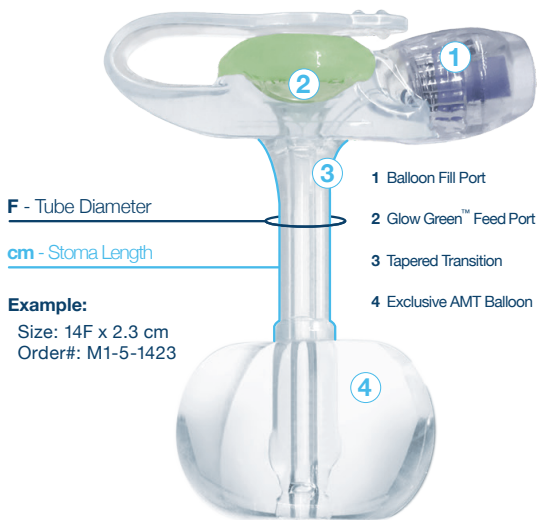

Example:

Size: 14F x 2.3 cm
Order#: M1-5-1423


MiniONE® also available in Non-Balloon Options:

MiniONE® Capsule Non-Balloon: 14F, 18F, 20F, 24F. MiniONE® Non-Balloon: 14F, 18F, 20F, 24F.

Feed Sets Included in Kit

LEGACY KIT

LEGACY – 8-1255

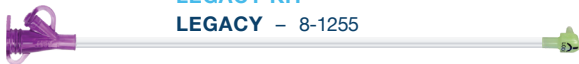

LEGACY – 8-1211




ENFit® KIT

ENFit® – 8-1255-ISOSAF



ENFit® – 8-1211-ISOSAF

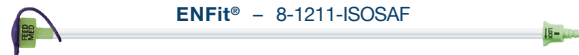

AppliedMedical.net | 800 869 7382 | CS@AppliedMedical.net | ICS@AppliedMedical.net



Äterförsäljare:
MEDI PLAST
Mediplast AB
Box 1004, 212 10 Malmö
T 020-78 80 35
info.se@mediplast.com
www.mediplast.com



Applied Medical Technology, Inc.
8006 Katherine Boulevard
Brecksville, OH 44141
An ISO 13485 Registered Company
MR Conditional | Made In USA



©2022 Applied Medical Technology, Inc.

Patents: <http://www.AppliedMedical.net/Patents/> – Trademarks: <http://www.AppliedMedical.net/Trademarks/> | C-4513-E

* The MiniONE® Balloon Button comes packaged as 1/box. | ENFit® is a registered trademark of GEDSA.