 MEDI PLAST[®]	
	
KoloBa™ SOFT BLACK EDITION	
Mediplast AB Kantyxegatan 29, SE-213 76 Malmö, Sweden Tel.: +46 40 671 23 00 / www.mediplast.com	  

EN

KoloBa™ SOFT Black Edition

1-piece, closed, colostomy pouch in black with divided front side, filter and an extra-soft skin barrier

Product description

KoloBa™ SOFT Black Edition is a closed pouch for colostomy with a carbon filter with a unique extra partition wall with air channels. This substantially reduces the risk of the filter becoming covered or clogged. KoloBa™ SOFT Black Edition has an extra-soft and thinner skinwafer for optimal adhesion and flexibility. The pouch is covered by a soft, black non-woven layer and has an openable front with an inspection window and comes in sizes Mini (100 ml), Midi (200 ml) and Maxi (300 ml) with both cuttable and pre-cut skin barriers. One box contains 30 KoloBa™ SOFT Black Edition pouches and filter cover that protects in the shower and bath.

Instructions

Adapting the hole in the skinwafer

To avoid leakage problems and irritated skin it is important that the skin wafer fit snugly around the stoma. If you are unsure of the stoma size, use a measuring guide to control the size. Use the cutting guide on the skin wafer and draw the stoma shape on the protective paper. Before cutting, blow some air into the pouch to separate the plastic layers, which decreases the risk of cutting into the pouch. Adjust the hole in the wafer using small curved scissors.

Preparing the skin

Wash the skin around the stoma with lukewarm water. Avoid oil-based soaps or creams since they may decrease the skin wafers adhesive properties. If soap is used it should be unscented.

Allow the skin to air-dry. The skin should be completely dry before attaching the skin wafer around the stoma.

Attaching the pouch

A body-temperature hydrocolloid adhesive always fastens best. Therefore, warm the skin wafer between your hands for a while before attaching it around the stoma. The pouch has an inspection window to make it easier to apply the skin wafer correctly around the stoma.

Remove the protective paper from the skin wafer and attach it to the skin around the stoma. Gently rub the skin wafer to make it attach it even better.

Removing the pouch

Start by dabbing with plenty of lukewarm water around skin wafers edges; this will saturate the hydrocolloid and make it easier to remove. Gently roll off the wafer downwards.

Avoid using oil, alcohol or solvents as it can affect how well the next skin barrier attaches to the skin

Waste handling

Used pouches are drained, packaged and dispose with household waste.

Warnings and precautions

Store in a cool and dry place, but never in the refrigerator.

Do not use if the packaging is damaged or after the expiry date.

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