 MEDI PLAST[®]

UroBa[®]
Mediplast AB Kantyxegatan 29, SE-213 76 Malmö, Sweden Tel.: +46 40 671 23 00 / www.mediplast.com

EN UroBa[®] 1-piece, ostomy pouch with a non-return valve, divided front side a 2-layer skin barrier
Product description
UroBa[®] is a drainable urostomy pouch with a built-in, non-return valve, so that the content does not flow back onto the stoma, and a soft drain fixture for easy emptying. UroBa[®] has a double-lay-er adhesive with a combination of two adhesive hydrocolloids; the one closest (the darker one) to the stoma is conformable with high absorption, and the perimeter (the lighter one) is thinner for optimum adhesion and flexibility.
T The pouch is covered by a soft, non-woven layer and has an openable front with an inspection window and comes in a size of 400 ml. UroBa[®] is available with both cuttable and pre-cut skin wafers.
One box contains 30 UroBa[®] pouches and night connectors.

Instructions
Adapting the hole in the skinwafer
To avoid leakage problems and irritated skin it is important that the skin wafer fit snugly around the stoma. If you are unsure of the stoma size, use a measuring guide to control the size. Use the cutting guide on the skin wafer and draw the stoma shape on the protective paper. Before cutting, blow some air into the pouch to separate the plastic layers, which decreases the risk of cutting into the pouch. Adjust the hole in the wafer using small curved scissors.
Preparing the skin
Wash the skin around the stoma with lukewarm water. Avoid oil-based soaps or creams since they may decrease the skin wafers adhesive properties. If soap is used it should be unscented.
Allow the skin to air-dry. The skin should be completely dry before attaching the skin wafer around the stoma.
Attaching the pouch
A body-temperature hydrocolloid adhesive always fastens best. Therefore, warm the skin wafer between your hands for a while before attaching it around the stoma. The pouch has an inspec-tion window to make it easier to apply the skin wafer correctly around the stoma Remove the protective paper from the skin wafer and attach it to the skin around the stoma. Gently rub the skin wafer to make it attach it even better.
Removing the pouch
Start by dabbing with plenty of lukewarm water around skin wafers edges; this will saturate the hydrocolloid and make it easier to remove. Gently roll off the wafer downwards.
Avoid using oil, alcohol or solvents as it can affect how well the next skin wafer attaches to the skin.
Emptying the pouch
Hold the pouch outlet away from the body and pull the plug that is attached to the outlet upwards. Then, lower the drain/outlet over the toilet and empty. Wipe off the outlet and reset the stopper. Make sure to press it all the way down to the bottom of the outlet.
Night connector
Each package contains connectors. These are used when connecting the pouch outlet to a night pouch. Before connecting the ostomy pouch, allow some liquid to remain in it in order to prevent negative pressure. Connect the night pouch to the narrower part of the connection piece and open the pouch outlet by pushing all the way down to the bottom of the wider part of the connector.

Waste handling
Used pouches are drained, packaged and dispose with household waste.
Warnings and precautions
Store in a cool and dry place, but never in the refrigerator.
Do not use if the packaging is damaged or after the expiry date.

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