



Medifusor - Pressure Infusion Bag

Pressure infusion bag is an inflatable cuff that is intended to apply pressure on I.V. bag to assist rapid fluid infusion.

- Available in nylon coated PU (Polyurethane) which fits both 500 ml and 1000 ml infusion bags, intended for single use
- Also available in transparent PU which also fits for 500 ml resp. 1000 ml infusion bags, intended for multiple use
- Operating pressure ranges 0~300mmHg
- Consult complete IFU before use

Art.No.	Description	Ward/Trsp.box
604805	Grey nylon 500-1000 ml	1/6
604807	Transparent 500 ml	1/6
604810	Transparent 1000 ml	1/6
Spare part		
604811	Pump with 3-v valve fits all pressure infusion bags	1/30

ward box = min qty / order



Cleaning and maintenanceMedifusor Pressure Infusion Bag for multiple use

1. Wipe the surface of cuff, tubing, and pressure bulb by 70% alcohol, avoiding the gauge. Do not spray alcohol so mist might immerse into the gauge and let it dry at room temperature after minutes. Confirm that the gauge shall be fully dry before storage.
2. Periodically check if the pointer of gauge moves with pumping the bulb a little, as well as if the pointer stays in zero when no pressure applies.

Material:

Art No 604805 Nylon coated PU

Art No 604807 Transparent PU

Art No 604810 Transparent PU

Tubes and pump in PVC

