Mediplast AB

Kantyxegatan 29, SE-213 76 Malmö, Sweden

Tel.: +46 40 671 23 00 / www.mediplast.com

**EN****Skin X® Absorbent**

Super-absorbent for the bandaging of gastrostomies

Product description

Skin X® Absorbent is intended for use in case of gastostomy leakage. Skin X® Absorbent is a plastic-coated super-absorbent to prevent soaking. It can be used separately, but intended to be placed over a Skin X® Protection ring to thus prevent irritating gastric fluids from leaking on the skin. Check the Skin X® Absorbent frequently and replace as often as needed, particular after meals.

Skin X® Absorbent is available in 4 hole sizes and with different absorption capacity; 3 and 7 mm (8 ml adsorption), 10 and 15 mm (10 ml adsorption).

One box contains 100 Skin X® Absorbent.

Instructions

Skin X® Protection ring protects the skin around gastostomy from leakage. Place the Skin X® absorbent on top of the ring to prevent gastric juices from leaking onto the skin. The hole in the barriers should fit as tightly as possible around the catheter/button. The barriers have a slit so they easily can be placed around a catheter or button. Secure with the supplied adhesive patch.

Preparing the skin

Wash the skin around the gastostomy with lukewarm water. Avoid oil-based soaps or creams since they may decrease the skin wafers adhesive properties. If soap is used it should be unscented. Allow the skin to air-dry before the Skin X® is applied.

Applying the bandage

A body-temperature adhesive always fastens best. Therefore, warm the skin barrier between your hands for a while before attaching it.

Remove the protective paper from the hydrocolloid on the back of the barrier, then slip it around the catheter/tubing and secure the skin protection ring to the skin.

It helps to massage the skin through the skin barrier to attach it even better. Start at the bottom and smooth upward with your fingers.

Place the absorbent on top of the skin barrier with the plastic-coated side out. Check Skin X® Absorbent frequently and replace as often as needed, particular after meals.

Removing the bandage

Start by dabbing the edges of the skin barrier thoroughly with lukewarm water, as this will saturate the hydrocolloid in the skin barrier and make it come off easier. Remove the bandage by gently rolling it off.

Avoid using oil, alcohol or solvents, as they can affect how well the next skin barrier will sit.

Waste handling

Used absorbents and hydrocolloid plates are with household waste.

Warnings and precautions

Store in a cool and dry place, but never in the refrigerator.

Do not use if the packaging is damaged or after the expiry date.

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