Codman's got you covered.

The global leader in neurosurgery is your one-stop, comprehensive source for proven, market-leading dural repair solutions.





The Leader in Dural Repair becomes your partner from Access to Closure

By expanding one of the broadest neurosurgical product lines, we are strengthening our commitment to providing you with what you need, when you need it.



DuraGen[®]Seci **Extended Tip Applicator** Bicol DuraGen[®]Plus **Codman**[®] **Disposable Perforators Codman**[®] Surgical Patties Brands You TRUST **Isocool**® DuraSe **Codman**[®] Surgical Strips **CustomBone Service**[™] **DuraSeal**[®]Xact Dural Sealant System **Codman VersaTru® MicroMyst** Suturable DuraGe (\mathbf{R}) Dural Regeneration Matrix

Now, It's All Here.



Codman Perforators and Craniotomy Kits

Codman Perforators have been the #1 choice of neurosurgeons for more than 30 years. The automatic disengagement mechanism and atraumatic tip design help ensure safety and protection against thin bone breaking or nicking of the dura.

Patties and Strips

X-ray detectable and lint free, Codman Surgical Patties and Strips provide neurosurgeons with gentle protection and absorbency, to meet the demands of delicate intracranial and spinal surgery.





Codman® Disposable Bipolar Forceps

For more than 50 years, Codman[®] has innovated in the field of electrosurgery. Our broad line of disposable bipolar forceps are available to accommodate your diverse neurosurgical procedures.

The Codman VersaTru[®] disposable non-stick bipolar forceps help you optimize your surgical performance in challenging situations and the Isocool[®] disposable non-stick bipolar forceps help you in procedures requiring precise heat control.

Our full selection of cranial and spinal dural repair solutions.



DuraGen Family

With more than 1400 patients studied and more published data than any other collagen graft*, DuraGen Dural Regeneration Matrix is a frequently used clinically proven graft for CSF leak preventation and dural repair.

Suturable DuraGen is a bilayer graft with enhanced strength and support, providing the ability to suture without losing the conformability and resorption you demand from a dural graft.

Dural Sealants

The DuraSeal[®] Family of sealants has been specifically designed for use as adjuncts to standard methods of dural repair, such as sutures, to provide watertight closure intraoperatively and through the critical healing period. DuraSeal[®] is more effective at preventing CSF leaks than fibrin glue¹¹





Biomimetic Skull Bone Reconstruction

CustomBone Service is a patient specific implant intended to replace bony voids in cranioplasty. CustomBone Service is made of a bio-mimetic material with a chemical composition and structure that resembles the mineral component of human bones. The specific bio-mimetic chemical composition combined with an elevated interconnected porosity are suitable for housing cells responsible for bone regeneration, playing an important role in the process of perimetral osteointegration of the prosthesis.

Comprehensive Coverage From Codman

The Leader in Dural Repair becomes your partner from Access to Closure

Everything your Neuro OR team needs for cranial and spinal access, dural repair and closure Codman is your single, comprehensive source.

nded Use: Cust

(for children above 2 years of age). "CustomBone" custom-made implants are suitable for reconstructing cranial defects resulting from: trauma and vascular pathologies either associated or non-associated to cranial decompression; removal of tumours; reabsorption of autologous bone; rejection of other prosthetic materials; congenital

malformations. Contraindications: Use of "CustomRone" prostheses should be avoided in the presence of inflammatory conditions and manifest infections of the surgical area to be treated, as well as in insufficiently vascularized sites. The "CustomRone" prostheses should not be implanted if dura mater lacerations are present, haddition, they should not be used in plantents sufficiently from provide synapsensitivity to aclium hoposphates or series from Soft and the soft and the provide structure structure and the soft and

DuraSeal Xact Indications: The DuraSeal® Xact Sealant System is indicated for use during spine procedures as an adjunct to standard methods of dural repair, such as sutures, to pr waterrught closure. Contraindications: Do not apply the DuraSeal[®] Xact Spinal Sealant in abdominopelvic surgical procedures for use as a sealant or adhesion barrier.

DuraSaal Indications: The DuraSeal[®] canial sealant system is intended for use as an adjunct to standard methods of dural repair, such as sutures, to provide watertight closure Contraindications: Do not apply the DuraSeal[®] Canial Sealant in abdominopelvic surgical procedures for use as a sealant or adhesion barrier.

Extended Tip Applicator Indications: The Extended Tip Applicator is intended for use in the simultaneous delivery of two non-homogenous solutions onto a surgical site. Contraindications: Do not use Extended Tip Applicator for other indications than ones provided in the instructions for use.

MicroMyst⁺ Applicator Indications: The MicroMyst⁺ Applicator is intended for use in the delivery of two non-homogenous solutions onto a su Contraindications: Do not use MicroMyst⁺ Applicator for other indications than ones provided in the instructions for use

DuraGen[®] Secure Indications: DuraGen Secure Dural Regeneration Matrix is indicated as an onlay graft for the repair and restoration of dural defects in cranial and spinal surgical procedure Calls and and ward within triowing. DuraGen[®] Secure matrix may be used to close dural defects follow Indications: DuraGen Secure Dural Regeneration Matrix is indicated as an onlay graft for the repair and restoration of dural defects in canali and spinal surgical procedures. DuraGen⁺ Secure matrix neadly conforms to the surface of the brain, spinal code and overlying tissues. DuraGen⁺ Secure matrix may be used to the dural defects following any traumatic injury, excision, retraction or shrinkage. DuraGen⁺ Secure matrix may be used to subgelment primary closure. DuraGen⁺ Secure matrix may be used in the followin procedures: -Canali Convoity: may be used to cover large defects following surgery, especially for dural loss from exision, contraction, retraction and/or shrinkage. -Bani Swelling: ntra-operative brain swelling or articipated postoperative swelling: -Posterior forsta Surgery. I General use as a dural graft, 2 joint decompression caniectomy and dur relases for infarts: _O. Posterior Informatic general approximation with shutess, § Ja as expansion loger between the dura and overling tissues. Contraindications: DuraGen⁺ Secure matrix is not designed, solid or interded for use except as described in the functionations of the used matrix with a work history of physensimitity to boxine devine matrix-sing size spinal barders for Use and scentrarior and usery with attraction and the prime size science in molecular size. Should be used in this cuation in infected regions. -Not recommended to cover dural defects involving matoria circles. -Not recommended for large defects at the childra barder of size restorement. skull base following surgery.

Suturable DuraGen[®] Indications: Sturable DuraGen[®] in indicated as a dural substitute for the repair and restoration of dural defects in cranial and spinal surgical procedures. Suturable DuraGen[®] matrix readity controms to the surface of the brain and overlying tissues. Suturable DuraGen[®] matrix may be used to done dural defects following traumatic injury, exicaino retraction or shrinkage. Surturable DuraGen[®] matrix may be used to supplement primary docume. Suturable DuraGen[®] matrix may be used to done dural defects following procedures: -Cran Convexity: may be used to cover large defects following surgers, especially for dural loss from existion; constraction, retraction and/or shrinkage. =Brain Swelling into-aprece brain swelling or anticipated postoperative swelling. +Posterior Fossa Surgery: 1) Ceneral use as a dural gal, 2) decompression craniectomy and dural release for infarcts, i.e.,

*Summary outcome statistics derived from the following clinical studies:

- 1. Danish SF, et al: Experience with acellular human dura and bovine collagen matrix for duraplasty after posterior fossa decompression for Chiari malformations. J Neurosurg Pediatrics 104:16-20, 2006 2. Harvey RJ, et al: Closure of large skull base defects after endoscopic transnasal craniotomy. J Neurosurg 111 (2): 371-379, 2009 3. Horaczek JA, et al: Collagen matrix in decompressive hemicraniectomy. Neurosurgry 63 (1 suppl.1): ONS 176-ONS 181, 2008
- 4. Lee JH, et al: Dural reconstruction in meningioma surgery in, Lee JH (ed): Meningiomas: Diagnosis, Treatment an Outcome. London: Springer, 2009, pp 619–624 5. Narotam PK, et al. A clinicopathological study of collagen sponge as a dural graft in neurosurgery. J Neurosurg 82:406–412, 1995.
- 6. Narotam PK, et al: Collagen matrix (DuraGen) in dural repair: analysis of a new modified technique. Spine 29 (24) :2861-2867, 2004
- Narotam PK, et al: Collagen matrix duraplasty for cranial and spinal surgery: a clinical and imaging study. J Neurosurg 106 (1):45–51, 2007
 Narotam PK, et al: Collagen matrix duraplasty for posterior fossa surgery: evaluation of surgical technique in 52 adult patients. J Neurosurg 111 (2):380-386, 2009
- 9. Sade B, et al: Non-watertight dural reconstruction in meningioma surgery: results in 439 consecutive patients and a review of the literature. Clinical article. | Neurosurg 114 (3):714-718, 2011. 10. Stendel R, et al: Efficacy and safety of a collagen matrix for cranial and spinal dural reconstruction using different fixation techniques. | Neurosurg 109 (2):215-221, 2008 11. Than KD, et al: Polyethylene glycol hydrogel dural sealant may reduce incisional cerebrospinal fluid leak after posterior fossa surgery. Neurosurgery 2008;63(suppl 1):ONS182-ONS186.

Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region.

- Non contractual document. The manufacturer reserves the right, without prior notice, to modify the products in order to improve their quality.
- Warning: Applicable laws restrict these products to sale by or on the order of a physician.
- Consult product labels and inserts for any indication, contraindications, hazards, warnings, precautions, and instructions for use.
- Products mentioned in this document are CE class III devices. Please contact Integra customer service should you need any additional information on devices classification. All the medical devices mentioned on this document are CE marked according to European council directive 93/42/EEC on medical devices and its relatives, unless specifically identified as "NOT CE MARKED".

Sales & Marketing EMEA

Integra LifeSciences Services (France) SAS Immeuble Séquoia 2 97 allée Alexandre Borodine Parc technologique de la Porte des Alpes

69800 Saint Priest = France

Phone: +33 (0)4 37 47 59 00 = +33 (0)4 37 47 59 99 (fax)

integralife.eu

Customer services EMEA

Europe, Middle-East, Africa: +33 (0)4 37 47 59 50 = +33 (0)4 37 47 59 25 (Fax) = csemea@integralife.com France: +33 (0) 437 47 59 10 = +33 (0) 437 47 59 29 (Fax) = custservfrance@integralife.com Benelux: +32 (0)2 257 4130 = +32 (0)2 253 2466 (Fax) = custsvcbenelux@integralife.com Germany: +49 (0) 2102 5535 6200 = +49 (0)2 102 553 6636 (fax) = custsvcgermany@integralife.com Italy : +39 (0)2 577 89 21 = +39 (0)2 575 113 71 = custsvcitaly@integralife.com Switzerland: +41 (0)2 27 21 23 30 = +41 (0)2 27 21 23 99 (Fax) = custsvcsuisse@integralife.com United Kingdom: +44 (0)1264 312 725 = +44 (0)1264 312 821 (fax) = custsvcs.uk@integralife.com

MicroMyst, DuraSeal, DuraGen, DuraGen Secure, DuraGen Plus, Codman, Integra, and the Integra logo are registered trademarks of Integra LifeSciences Corporation or its subsidiaries in the United States and/or other countries. UltraPure Collagen is trademarks of Integra LifeSciences Corporation. CustomBone Service is a trademark of Fin-Ceramica Faenza S.p.A ©2018 Integra LifeSciences Corporation. All rights reserved. Last revision date: 05/2018. 0935429-2-EN DOCUMENT FOR USE IN EUROPE, MIDDLE-EAST and AFRICA.

A large direct sales force specialized in neurosurgery and dedicated to hospital in-service and product training

Posterior Inferior Careballar Artery (PICA) infarcts, 2) anticipated swelling aler trauma, and a) may be used in Chiari decompression procedures; -Spinal Surgery: as a spinal dural gra1, especially useful for defects arising from pinhole tears, disc surgery, and spinal stenosis decompression, 2) aler resection of intradural turnc aler dural approximation with sturnes, and 4) as a separation layer between the dura and overlying tissues. Contraindications: Suruble DuraCen[®] to not designed, sold to intended for use exocut a described in the indications for use and is contraindicated in the folio +For patients with a known history of hypersensitivity to bovine derived materials. - Should be used with caution in infected regions.

And the set of the set

CODMAN* Surgical Patties CODMAN* Surgical Strips Indications: The surgical patties and surgical strips are indicated for the use in protection of tissue, including brain and other tissues of the central nervous system, during surgery. Contraindications: Do not use for other indications than the ones specified in the instructions for use

CODMAN* Disposable Perforator Indications: The CODMAN Disposable Perforator is for use in perforating the cranium. When properly used, it is designed to automatically disengage once perforation is arromolished and when pressure is removed from the drill point.

BIGOL*Collegen Sponge Indications: The BICOL Collegen Sponge is for use in neurological procedures requiring prolonged netraction and exposure of the brain. The MICOL Collegen Sponge should not be used in those patients involve to be enrotive to calgue. The BICOL Collegen Sponge should not be pact into avocater should net, neurol texts, interfacial planes, sincers, foramine, or assess of confined attractional iterative. If confined packing should become necessary in o situations, care should be taken to avoid the possible development of undue pressure on vital structures.

CODMAN VersaTin" Indications. Codman "VersaTin" Standard and Slim Disposable Non-Stick Bipolar Forceps are intended for use in electrosurge Contraindications. Reterilization of used product by the custome. Use for contraceptive coagulation of fallopian tube tissue rgery for coagulation of tissue

JBOCOOP Indications: The Isocool® Bipolar Forceps (handles and tips) when used as part of a system including a bipolar electrosurgical generator are indicated for c coagulating, grasping and manipulating tissue during general surger, neurosurgery, ENT surgery, OB/C/N surgery, and maxillofacial/ plastic surgery proc Bipolar Forceps are also indicated for cauterizing, coagulating, grasping and manipulating soft tissue during spine surgery and orthopedic surgery. Contraindications: Use for contraceptive coagulation of fallopian tube tissue. dicated for cauterizing,





EC REP Codman Pinewood, Campus, Nin Mile Ride

Raynham, MA 02767-0350 = USA

CE CE 0123 0086 TÜV BSI

Wokingham, RG403EW = United Kingdom Integra LifeSciences Corporation 311 Entreprise Drive

Plainsboror, NJ = 08536 USA

EC REP Integra LifeSciences Services (France) SAS Immeuble Séquoia 2 97 allée Alexandre Borodine Parc technologique de la Porte des Alpes 69800 Saint Priest = France

Manufacturing Warehouse for Codman systems

Codman & Shurtleff, Inc = 325 Paramount Drive