


MiniONE[®] Button

Low Profile Balloon Gastrostomy Tube

LEGACY + ENFIT[®]


Replacing AMT MiniONE[®] Balloon Button*

To ensure a proper fit, it is crucial to measure the stoma length using the AMT Balloon Stoma Measuring Device (BSMD). This is not a substitute for the DFU/IFU (Directions for Use/Instructions for Use). [Icon]

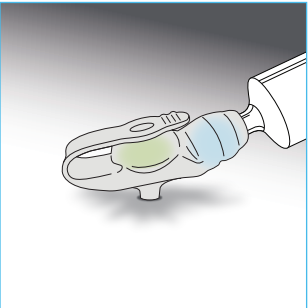


DEFLATE BALLOON

Deflate the existing device using a Luer syringe or Luer Lock syringe.

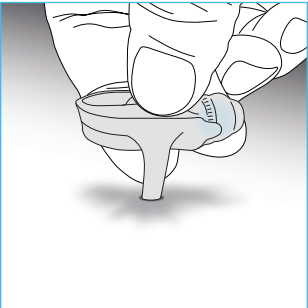


REMOVE DEVICE

Remove the device, gently pull up on the device with slow and steady pressure.

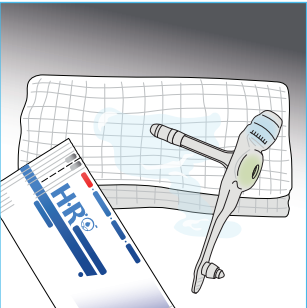


PREPARE KIT

Prepare the new device, apply a water soluble lubricant to the device.

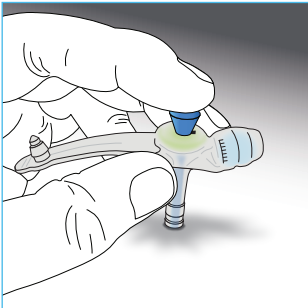

INSERT DEVICE

Insert the device into the stoma site, until it is flush against the skin. Remove the blue stiffener.

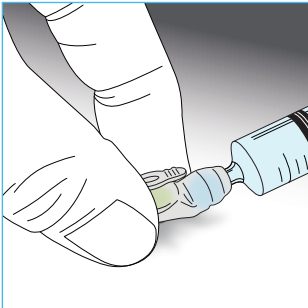


FILL BALLOON

Fill the balloon to the recommended fill volume, which is printed on the balloon fill valve.


Attaching an AMT Feed Set

The MiniONE[®] Balloon Button and corresponding feed set feature our patented Glow Green[™] Technology.

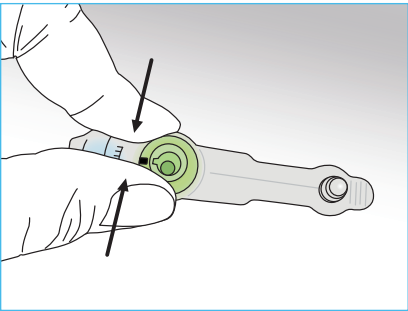


STABILIZE

Stabilize the MiniONE[®] device.

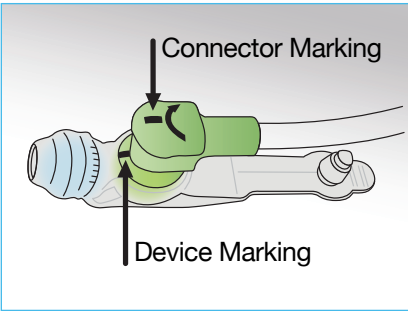


ALIGN + INSERT

Align the marking lines and insert the connector into the MiniONE[®].

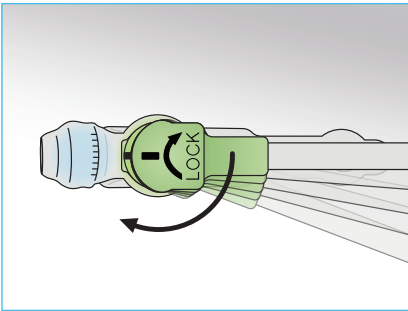

ROTATE

Rotate connector 3/4 to the right.

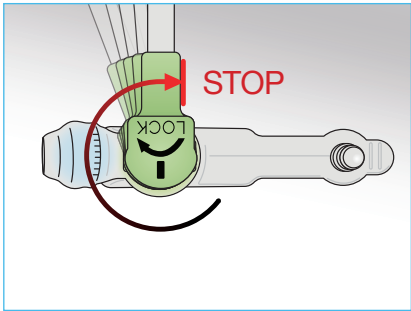

LOCK

Lock the connector into place, now the device is ready to begin feeding.

Legacy + ENFit[®]

Available MiniONE[®] Balloon Button Sizes: 12F – 24F



F - Tube Diameter

cm - Stoma Length

Example:


Size: 14F x 2.3 cm
Order#: M1-5-1423

- 1 Balloon Fill Port
- 2 Glow Green[™] Feed Port
- 3 Tapered Transition
- 4 Exclusive AMT Balloon

Feed Sets Included in Kit

LEGACY KIT

LEGACY – 8-1255

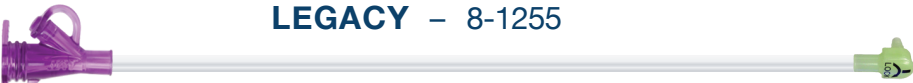


LEGACY – 8-1211



ENFIT[®] KIT

ENFit[®] – 8-1255-ISOSAF

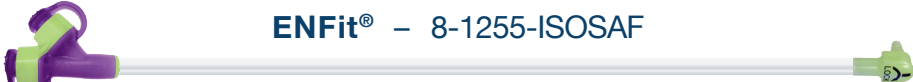

ENFit[®] – 8-1211-ISOSAF



Questions? Call Today:

Jälleenmyyjä:

MEDI PLAST

Medioplast Fenno Oy

PL 153, 01531 Vantaa

Puh: 09 276 360

asiakaspalvelu@mediplast.com

www.mediplast.com

MiniONE[®] also available in Non-Balloon Options:

MiniONE[®] Capsule Non-Balloon: 14F, 18F, 20F, 24F

MiniONE[®] Non-Balloon: 14F, 18F, 20F, 24F



For a complete list of products visit us at www.AppliedMedical.net

800 869 7382 | CS@AppliedMedical.net | ICS@AppliedMedical.net

© 2022 Applied Medical Technology, Inc. C-4627-B
Patents: <http://www.AppliedMedical.net/Patents/> | Trademarks: <http://www.AppliedMedical.net/Trademarks/>
*The MiniONE[®] Balloon Button comes packaged as 1/box. | ENFit[®] is a registered trademark of GEDSA.

